



The freedom to move

Forearm Winglet Foam Set Instructions

Purpose:

Both the Midi and Maxi cuff circumference are identical, so to reduce the diameter of the forearm support cuff for either forearm winglets may be inserted. This may be necessary for people with narrower forearms who want a more snug fit.

Warning!

Exchanges and returns will not be accepted once the wingtips have been permanently adhered to the plastic. Please first set them up and check they fit before doing this.

1. First try the smartCRUTCH for size (bear in mind, bulky Winter clothing could be a factor.)
2. Test-fit the first piece as shown in Fig. 1 with the white adhesive cover placed on the inside of the cuff and the winglet foam aligned with the existing foam.
3. Once you are satisfied, remove the white adhesive cover (Fig. 2) and stick in place.
4. Repeat with the remaining 3 pieces.

